


Initial: 10/10/2007
 Reviewed/revised:
 Revision:

MILWAUKEE COUNTY EMS
 PRACTICAL SKILL
SERIAL CABLE DATA
UPLOAD FOR ZOLL M-SERIES

Approved by: Ronald Pirrallo, MD, MHSA
 Signature:
 Page 1 of 1

Purpose: To transfer ECG and resuscitation information from the Zoll M-series monitor to the RescueNet server using a serial cable		Indications: Patients with any Zoll M-series monitoring	
Advantages: Captures and analyzes all monitoring, CPR, capnography information electronically	Disadvantages: None	Complications: Loss of information if upload procedure not followed correctly	Contraindications: None

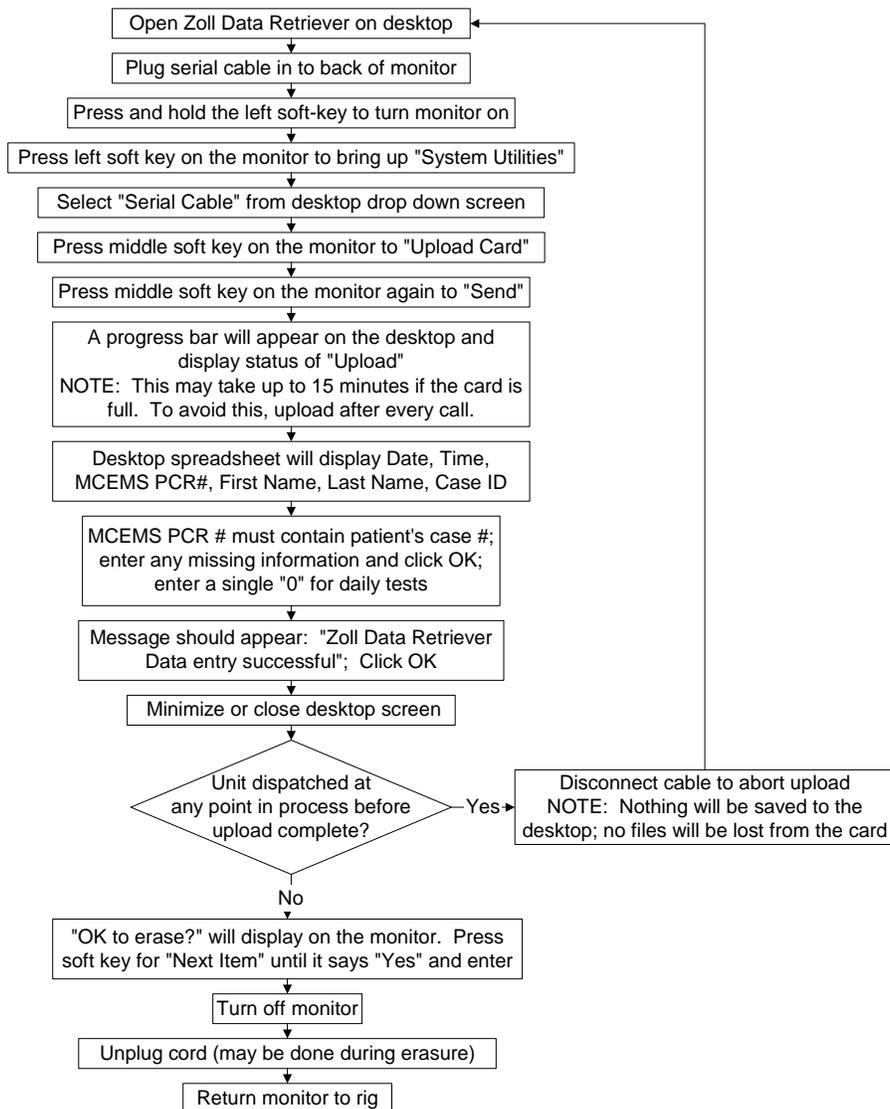

NOTES:

- The MC EMS PCR number must be entered for every case to link the ECG information to the patient's electronic run report. The number can be entered at any time – during the call or at the time of upload
- Enter a single "0" as the MC EMS PCR number for daily tests
- To avoid entering the PCR number numerous times, leave the monitor on and leads attached to the patient during the entire call