



ZOLL LIVESTREAM

EMT-Basic

Advanced EMT

Paramedic

PROCEDURE

- ❑ Ensure your ZOLL device is connected to your network -- network symbol will illuminate
- ❑ Press arrow soft key on left side of monitor to display tier two key choices
- ❑ Press Livestream soft key on left side of the monitor
- ❑ Confirm on-screen prompt to begin streaming
- ❑ To discontinue live streaming, repeat above steps to cancel

*Revised-
focused on
providers*

REFERENCE GRAPHICS

Ensure you are connected
to your network



KEY POINTS

- **Livestream should be initiated for ALL OLMC consults**
- If problems are encountered connecting to Livestream, check the monitor's Bluetooth or WiFi network
- All system devices are set up for Livestreaming

Medical Director: Ben Weston, MD, MPH

Date: April 2023