



# X-Series Live Stream

Supervisor/Physician

## INDICATIONS

- Any patient that requires online medical control (OLMC) consultation where live streaming would be beneficial for patient care


## PURPOSE

- Provide live stream to OLMC physician of patient data to improve patient care

## CONTRAINDICATIONS

- No defined purpose for OLMC consultation

## PROCEDURE

- Establish immediate radio contact with desired EMS unit per standard OLMC process
- Discuss the clinical utility and availability of the Live Stream with the EMS unit on scene
- If useful and available, proceed to next steps. If not useful or not available, return to standard OLMC process
- Proceed to: milwaukee.zollstreaming.com; enter username (mkeuser) and password (if not autofilled) and sign in
- Select the desired device to connect to by using the "Unit id" and "Department id" provided by the EMS unit
- Live streaming monitor data will now be displayed on your screen
- Select the  icon to create reports and view previously performed 12-lead ECGs

## REFERENCE GRAPHICS



# ZOLL


Welcome to *Mobile Streaming*  
Visit [ZOLL.com](http://ZOLL.com) to learn more about ZOLL products

Username  
mkeuser

Password  
\*\*\*\*\*

Sign In

© Copyright 2020 All rights reserved.  
ZOLL Medical Corporation

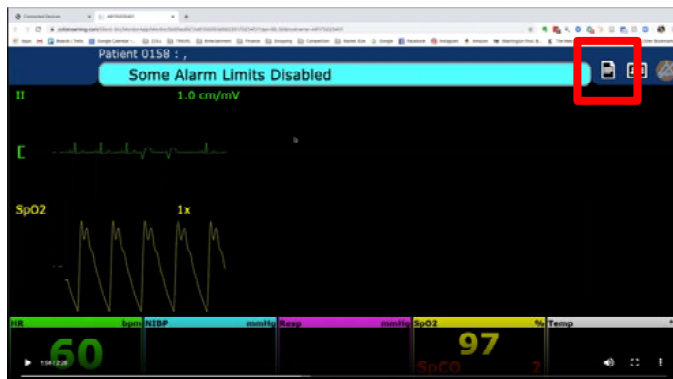

## KEY POINTS

- Reminder:** Only one remote viewer can access Live Stream concurrently
- Reminder:** You can only view data; you cannot control the Zoll X-Series remotely
- Username: mkeuser
- Complete the [Feedback Form](#)



Medical Director: Benjamin Weston, DO, MPH, FACEP

April 2020